

# FAQ - Oncologist

## Q. Which dose should be used?

- A. Maximum recommended total dose of doxorubicin per procedure is 150mg. PRECISION I included a dose escalation study up to 150mg doxorubicin and the 150mg dose was safe and effective. There were no dose-related adverse events. The tumour response rate at 6 months was 87% at this dose.

In Precision V, Biocompatibles' phase III randomised controlled clinical trial, the dose is 150mg.

A dose of up to 37.5mg doxorubicin per ml DC Beads can be loaded. 150mg in 4 to 6ml DC Bead.

